


Alpha Stand-up Support Magic & Standard


Always
ready
to support

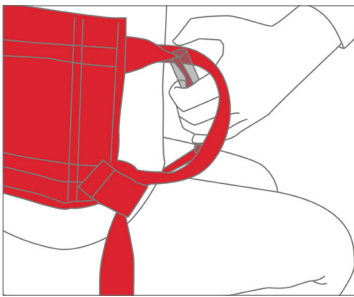

The Alpha Stand-up Support is a stable back strap. With this supportive product caregivers can securely hold on to and guide clients when standing up, sitting down and during transfers.

When being used, the simple handling can easily be combined with a back-friendly work methods. The Alpha Stand-up Support Magic in contrast to the Standard has an adjustable strap, which adapts to the clients' need.

In the illustrations below you will see the light blue PM-3005, the Alpha Leg Support & Extension, which is a great aid to hold legs together which stabilizes the clients in their position during transfers.

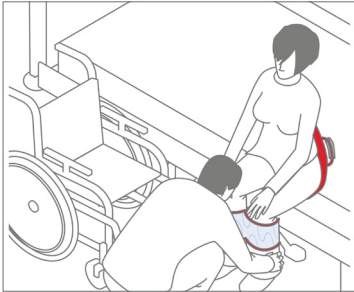

Available as an accessory to the Standard is the Alpha Looped Belt, PM-3010, which enables the safe transfer for clients who sit in narrow (wheel)chairs. Both loops of the Looped Belt must be attached to the grips of the holding belt and with help of the clip locks then applied under the client's legs.

Technical specifications



Illustration with both PM-3012, PM-3015 & PM-3005

Product Name	Art. nr.	Dimensions	Material	Max lifting weight
Alpha Stand-up Standard	PM-3012	60 x 20 cm 23,6" x 7,8"	Polyamide, latex free	200 kg / 440 lb
Alpha Stand-up Magic	PM-3015	60 x 20 cm 23,6" x 7,8"	Polyamide, latex free	200 kg / 440 lb
Accessory to PM-3012: Alpha Looped Belt			PM-3010	



PETERMANN
part of the Human Care Group

These products are distributed by
Human Care AB for Petermann GmbH.

Visit www.humancaregroup.com for the latest product information and contact details to national offices and partners.